**TITAN-PORT**     
      

## TITAN-PORT GROSS

Implantierbare Port-Kathetersysteme

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**TITAN-PORT**    <sup>®</sup>  
      

## TITAN-PORT SMALL

Implantable port catheter systems

### Indications:

- systemic chemotherapy
- infusion therapy and long-term parenteral nutrition
- pain therapy
- long-term venous access (e.g. HIV-patients)
- administration of medication
- regional chemotherapy (e.g. liver tumors or metastases)
- blood sampling, poor vascular situation

Art. No.	Port						Catheter					PZN
	Material	Base	Height	Septum Ø	Weight	Reservoir	Material	OD	ID	Length	French	
111244 P-N paediatric (venous)	Titan	20 mm	9,5 mm	8 mm	<10g	0,4 ml	Poly-urethan	0,95 mm	0,75 mm	120 mm	3,0	01673103
111245 P-SET *paediatric (venous)	Titan	20 mm	9,5 mm	8 mm	<10g	0,4 ml	Poly-urethan	1,7 mm	1,0 mm	700 mm	5,15	00199289
111251 A (arterial)	Titan	20 mm	9,5 mm	8 mm	<10g	0,4 ml	Poly-urethane**	1,7 mm	1,0 mm	700 mm	5,15	01673793
111254 S (peridural)	Titan	20 mm	9,5 mm	8 mm	<10g	0,4 ml	Poly-urethane	0,85 mm	0,45 mm	900 mm	2,58	01673801

\* SET including introducer set and tunnelizer  
 \*\* with fixation rings

### Advantages of Titanium

- ✓ excellent biocompatibility
- ✓ non magnetic
- ✓ hypoallergenic
- ✓ no chemical reactions
- ✓ light weight
- ✓ MRI compatible 

### Product specifications:

- ✓ catheter with additional side-hole
- ✓ self-sealing silicone membrane
- ✓ easy and safe screw-connection between port chamber and catheter
- ✓ pressure stability
- ✓ sufficient flow and persistence only with the corresponding special cannulas

example

example

example

### Set standard:

1 Titan-Port • 1 catheter • 1 special puncture needle  
 1 catheter introducer • 1 rinsing needle  
 (suitable introducer sets are available separately, see accessories)

### Set complete:

1 Titan-Port • 1 catheter • 1 special puncture needle  
 1 catheter introducer • 1 rinsing needle  
 1 introducer set • 1 tunnelizer

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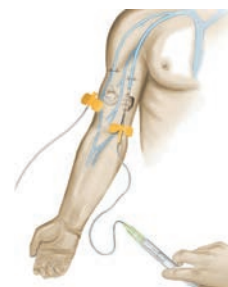
**TITAN-PORT**

## TITAN-PORT UA (SIDE PORT)

Implantable port catheter systems

### Indications:

- systemic chemotherapy
- infusion therapy and long-term parenteral nutrition
- long-term venous access (e.g. HIV-patients)
- administration of medication
- pain therapy
- blood sampling, poor vascular situation
- regional chemotherapy

Art. No.	Port					Catheter					PZN
	Material	Height	Length	Width	Weight	Material	OD	ID	Length	French	
111258 V-UA (venous)	Titan	8 mm	25 mm	14 mm	<5g	Silicone	2,0 mm	1,0 mm	700 mm	6	00272514
<b>Recommended port puncture needle straight</b>											
Art. No.	Canulae			Extra							PZN
	Ø	Length	Gauge								
SFN 0750 G	0,7	50 mm	22	Straight with Luer-Lock (without tubing)							00560673
SFN 0930 G	0,9	30 mm	20	Straight with Luer-Lock (without tubing)							01060286
SFN 0950 G	0,9	50 mm	20	Straight with Luer-Lock (without tubing)							03425332

### Also suitable

Art. No.	Port						Catheter					PZN
	Material	Base	Height	Septum Ø	Weight	Reservoir	Material	OD	ID	Length	French	
111245 P-SET* paediatric (venous)	Titan	20 mm	9,5 mm	8 mm	<10g	0,4 ml	Poly-urethane	1,7 mm	1,0 mm	700 mm	5,15	00199289
<b>Needles recommended for puncture</b>												
Art. No.	Canulae			Extra							PZN	
	Ø	Length	Gauge									
SFN 0920 S	0,9	20 mm	20	curved, with tubing and clamp							06910275	

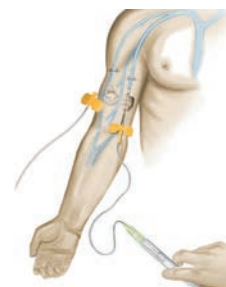
\* SET including introducer set and tunnelizer

### Advantages of Titanium

- ✓ excellent biocompatibility
- ✓ non magnetic
- ✓ hypoallergenic
- ✓ no chemical reactions
- ✓ light weight
- ✓ MRI compatible

### Product specifications:

- ✓ catheter with additional side-hole
- ✓ self-sealing silicone membrane
- ✓ easy and safe screw-connection between port chamber and catheter
- ✓ pressure stability
- ✓ sufficient flow and persistence only with the corresponding special cannulas

**TITAN-PORT** <sup>AV</sup> <sup>D</sup> <sup>P</sup>®  
<sup>F</sup> <sup>G</sup> <sup>S</sup> <sup>PD</sup> <sup>PC</sup> <sup>APH</sup> <sup>AS/PT</sup>

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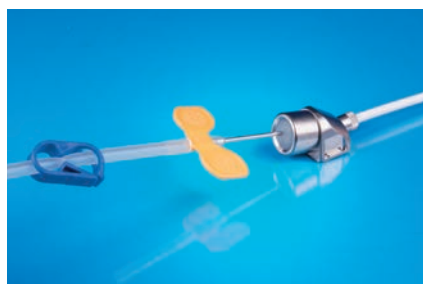
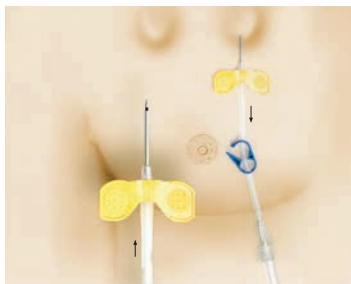
**TITAN-PORT**     
      

## TITAN-PORT® D (DIALYSIS)

Implantable port catheter system  
for venous-venous  
hemodialysis – single-lumen – double-lumen

### Indications:

- Venous access in kidney failure
- long-term venous access
- extracorporeal apheresis

Art. No.	Port						Catheter				PZN
	Material	Width	Length	Height	Septum Ø	Weight	Volume	Material	French	Volume	
111252 D-PU hemodialysis	Titan	24 mm	36 mm	17,5 mm	12 mm	<25g	0,6 ml	PU double-lumen	14	long portion: 2,3 ml short portion: 2,2 ml	01674462
including introducer set and tunnelizer											
111255 D hemodialysis	Titan	24 mm	36 mm	17,5 mm	12 mm	<25g	0,6 ml	Silicone single-lumen	12	catheter 60cm: 2,4 ml	01674166
including introducer set and tunnelizer											

### Special port needles

Puncture of port chamber only with special port needles is mandatory!

Art. No.	Canulae								PZN
	Ø	Length	Gauge	Volume	Flow	Puncture frequency	Unit	Extra	
DPK 1835*	1,8 mm	35 mm	15	1,0 ml	while hemodialysis ~ 300 ml/min in gravity 225 ml/min	~ 1000	25 Pcs.	with tubing	05108104
SFN 1835 G*	1,8 mm	35 mm	15	1,0 ml	while hemodialysis ~ 300 ml/min, in gravity 225 ml/min	~ 1000	25 Pcs.	straight with Luer-Lock-Adapter (optional with tubing)	10838997
DPK 2035	2,0 mm	35 mm	14	0,27 ml	while hemodialysis ~ 500 ml/min in gravity 300 ml/min	~ 500	25 Pcs.	with mandrin + needle free adapter	02429581

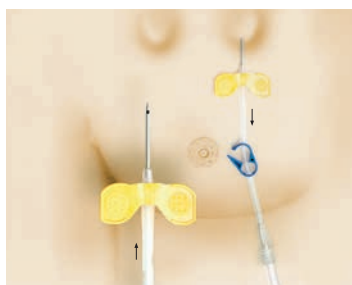
\*tubing contains PVC + DEHP

### Advantages of Titanium

- ✓ excellent biocompatibility
- ✓ non magnetic
- ✓ hypoallergenic
- ✓ no chemical reactions
- ✓ light weight
- ✓ MRI compatible 

### Product specifications:

- ✓ with single- or double-lumen catheter
- ✓ self-healing silicone membrane
- ✓ easy and safe screw-connection between port chamber and catheter
- ✓ pressure stability
- ✓ sufficient flow and persistence only with the corresponding special cannulas

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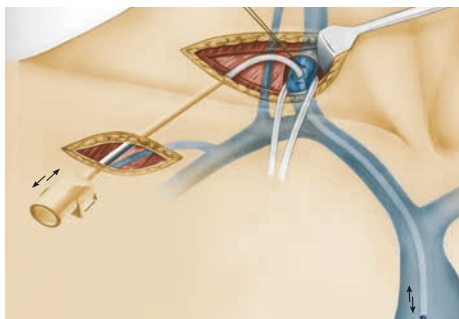
**TITAN-PORT**    <sup>®</sup>  
      

## TITAN-PORT APH

Implantable Port catheter system  
for extracorporeal apheresis

### Indications:

- long-term, high-volume venous access to central vascular system

Art. No.	Port						Catheter				PZN
	Material	Width	Length	Height	Septum Ø	Weight	Volume	Material	French	Volume	
111260 APH* extracorporeal apheresis	Titan	24 mm	36 mm	17,5 mm	12 mm	22 g	0,6 ml	Silicone	12	60 cm: 2,9 ml	10216652

\*including introducer set and tunnelizer

### Recommended port cannulas

Puncture of portchamber only with special port needles is mandatory!

Art. No.	Canulae								PZN
	Ø	Length	Gauge	Volume	Flow	Puncture frequency	Unit	Extra	
SFN 1320 S	1,3 mm	20 mm	18		72 ml/min	~ 3000	25 Pcs.	with tubing	07746257
SFN 1325 S	1,3 mm	25 mm	18		72 ml/min	~ 3000	25 Pcs.	with tubing	11094033
SFN 1535 G*	1,5 mm	35 mm	17	1,0 ml	while hemodialysis 200 ml/min, in gravity 150 ml/min	~ 1500	25 Pcs.	straight with Luer-Lock-Adapter (optional with tubing)	10216563
DPK 1835*	1,8 mm	35 mm	15	1,0 ml	while hemodialysis 300 ml/min in gravity 225 ml/min	~ 1000	25 Pcs.	with tubing	05108104
SFN 1835 G*	1,8 mm	35 mm	15	1,0 ml	while hemodialysis 300 ml/min, in gravity 225 ml/min	~ 1000	25 Pcs.	straight with Luer-Lock-Adapter (optional with tubing)	10838997
DPK 2035	2,0 mm	35 mm	14	0,27 ml	while hemodialysis 500 ml/min in gravity 300 ml/min	~ 500	25 Pcs.	with mandrin + needle free adapter	02429581

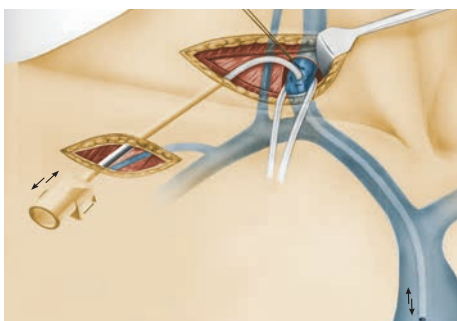
\* tubing contains PVC + DEHP

### Advantages of Titanium

- ✓ excellent biocompatibility
- ✓ non magnetic
- ✓ hypoallergenic
- ✓ no chemical reactions
- ✓ light weight
- ✓ MRI compatible 

### Product specifications:

- ✓ with single-lumen catheter
- ✓ self-healing silicone membrane
- ✓ easy and safe screw-connection between port chamber and catheter
- ✓ pressure stability
- ✓ sufficient flow and persistence only with the corresponding special cannulas

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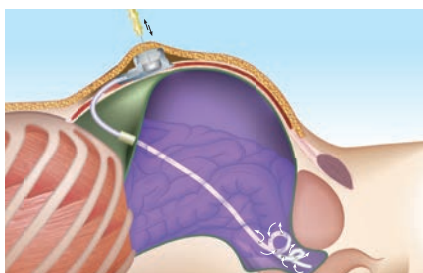
### Indications:

- Peritoneal dialysis
- local abdominal chemotherapy and other applications

### TITANPORT PD (peritoneal dialysis)

Art. No.	Port						PZN
	Material	Base	Height	Septum Ø	Weight	French	
111253 PD Peritoneal dialysis	Port: Titan Catheter: Silicone	29 mm	18 mm	14 mm	<18 g	14,5	01674172

### TITANPORT PC (peritoneal chemotherapy)

Art. No.	Port						PZN
	Material	Base	Height	Septum Ø	Weight	French	
111256 PC Peritoneal chemotherapy	Port: Titan Catheter: Silicone	29 mm	18 mm	14 mm	<18 g	14,5	01674321

**Port needles** Puncture of port chamber only with special port needles is mandatory e.g.

Art. No.	Canulae					PZN
	Ø mm	Length	Gauge	Unit	Extra	
DPK 2035	2,0	35 mm	14	25 Pcs.	with mandrin + needle free adapter	02429581
DPK 1835 *	1,8	35 mm	15	25 Pcs.	with tubing	05108104
SFN 1835 G*	1,8	35 mm	15	25 Pcs.	straight with Luer-Lock-Adapter (optional with tubing)	10838997
SFN 1535 G*	1,5	35 mm	17	25 Pcs.	straight with Luer-Lock-Adapter (optional with tubing)	10216563
or other high flow SFN®-port needles e.g.						
SFN 1320 S	1,3	20 mm	18	25 Pcs.	with tubing	07746257
SFN 1325 S	1,3	25 mm	18	25 Pcs.	with tubing	11094033

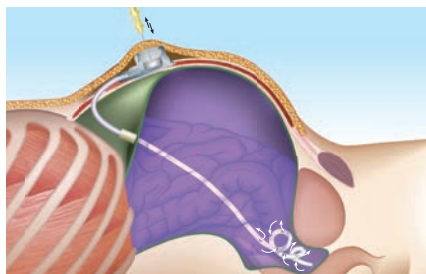
\* tubing contains PVC + DEHP

### Advantages of Titanium

- ✓ excellent biocompatibility
- ✓ non magnetic
- ✓ hypoallergenic
- ✓ no chemical reactions
- ✓ light weight
- ✓ MRI compatible

### Product specifications:

- ✓ catheter with perforations
- ✓ self-healing silicone membrane
- ✓ easy and safe screw-connection between port chamber and catheter
- ✓ pressure stability
- ✓ sufficient flow and persistence only with the corresponding special cannulas

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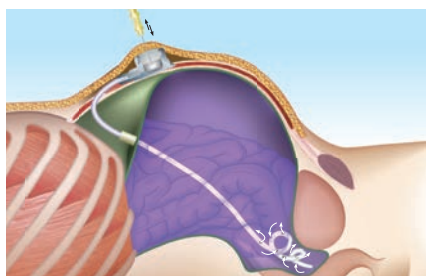
**TITAN-PORT**

## TITAN-PORT AS/PT

Implantable port catheter system for  
ascites puncture and  
peritoneal chemotherapy

### Indications:

- ascites puncture
- peritoneal chemotherapy

Art. No.	Port						Catheter			PZN
	Material	Base	Height	Septum Ø	Weight	Reservoir	Material	French	Reservoir	
111259 AS/PT Ascites- and peritoneal therapy port	Titan	29 mm	15,5 mm	14 mm	18 g	1,0 ml	Silicone	14,5	2,5 ml	10216623

### Recommended port cannulas

Puncture of port chamber only with special port needles is mandatory!

Art. No.	Canulae								PZN
	Ø mm	Length	Gauge	Volume	Flow	Punktions- häufigkeit	Unit	Extra	
SFN 1320 S	1,3	20 mm	18		72 ml/min	~ 3000	25 Pcs.	with tubing	07746257
SFN 1325 S	1,3	25 mm	18		72 ml/min	~ 3000	25 Pcs.	with tubing	11094033
SFN 1535 G*	1,5	35 mm	17	1,0 ml	in gravity 150 ml/min	~ 1500	25 Pcs.	straight with Luer-Lock-Adapter (optional with tubing)	10216563
DPK 1835*	1,8	35 mm	15	1,0 ml	in gravity 225 ml/min	~ 1000	25 Pcs.	with tubing	05108104
SFN 1835 G*	1,8	35 mm	15	1,0 ml	in gravity 225 ml/min	~ 1000	25 Pcs.	straight with Luer-Lock-Adapter (optional with tubing)	10838997
DPK 2035	2,0	35 mm	14	0,27 ml	in gravity 300 ml/min	~ 500	25 Pcs.	with mandrin + needle free adapter	02429581

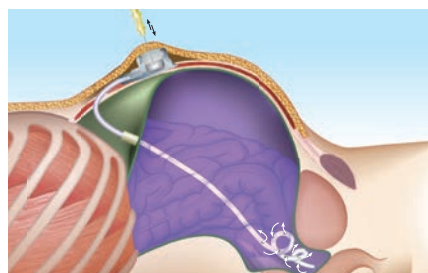
\* tubing contains PVC + DEHP

### Advantages of Titanium

- ✓ excellent biocompatibility
- ✓ non magnetic
- ✓ hypoallergenic
- ✓ no chemical reactions
- ✓ light weight
- ✓ MRI compatible

### Product specifications:

- ✓ catheter with perforations
- ✓ self-healing silicone membrane
- ✓ easy and safe screw-connection between port chamber and catheter
- ✓ pressure stability
- ✓ sufficient flow and persistence only with the corresponding special cannulas

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Implantable port catheter system

### Indications:

- long-term access to the central vascular system
- systemic chemotherapy
- admission of medication
- regional chemotherapy
- pain therapy
- blood sampling, poor vascular status
- infusion therapy and long-term parenteral nutrition

Art. No.	Port					Catheter					PZN
	Material	Base	Height	Septum Ø	Weight	Material	OD	ID	Length	French	
111248 G * (venous)	Titan	28 mm	13,5 mm	12 mm	<16 g	Silicone	2,5 mm	1,2 mm	700 mm	7,58	06149298

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**Patented INNOVATION**

ONE FOR ALL NEEDLES  
ALL FOR ONE PORT

PORT PUNCTURE NOW POSSIBLE WITH ALL NEEDLES!

- ⊕ cost savings by using "normal" needles for puncture
- ⊕ technical failures excluded → safety increased
- ⊕ no punch defects in the port membrane

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### Product specifications:

- ✓ specialty: no more obligation for special port needles!
- ✓ self-healing silicone membrane
- ✓ easy and safe screw-connection between port chamber and catheter
- ✓ pressure stability
- ✓ catheter with additional side-hole and length markings
- ✓ optimized design
- ✓ easy handling
- ✓ long-standing high-level quality

### One Port for all needles

- technical failures excluded » safety increased
- cost savings by using „normal“ needles for puncture
- no punch defects in the port membrane

### Set consists of:

- 1 Titan-Port + screw
- 1 single-lumen catheter
- 1 special puncture needle
- 1 introducer set
- 1 tunnelizer

**TITAN-PORT** <sup>AV</sup> <sup>D</sup> <sup>P</sup>®  
<sup>F</sup> <sup>G</sup> <sup>S</sup> <sup>PD</sup> <sup>PC</sup> <sup>APH</sup> <sup>ASPT</sup>

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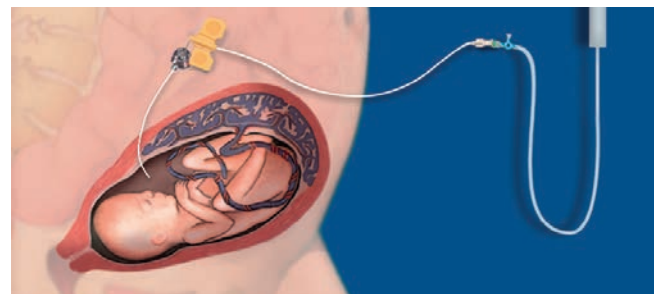
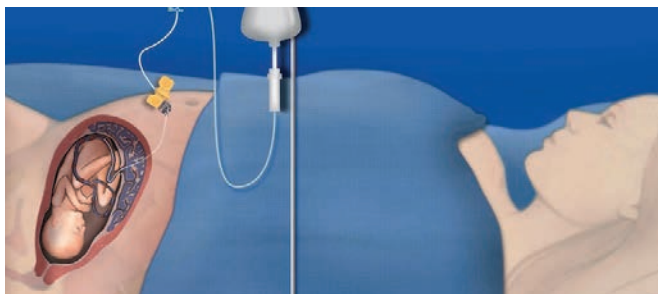
### Indications:

#### FETAL NUTRITION PORT-SYSTEM

- intrauterine growth restriction caused by placental insufficiency (IUGR)

#### AFR-PORT (Amniotic Fluid Replacement PORT-SYSTEM)

- preterm premature rupture of membranes with Anhydramnion (PPROM)

Art. No.	Port					Catheter					PZN
	Material	Base	Height	Septum Ø	Weight	Material	OD	ID	Length	French	
111243 F (fetal)	Titan	20 mm	9,5 mm	8 mm	<10 g	Polyurethane	0,64 mm	0,42 mm	400 mm	1,92	08830327
Recommended port cannulas for puncturing											
Art. No.				Canulae					PZN		
	Ø	Length	Gauge	Extra							
SFN 0720 S	0,7	20	22						07781615		

### FETAL

#### Contraindication:

posterior located placenta

#### Purpose:

improvement of fetal growth, prolongation of pregnancy

#### Usage:

long-term administration of nutriments into the umbilical vein such as aminoacids and glucosis via the port system

#### Period:

during 24-32nd week of gestation with IUGR (Intrauterine Growth Restriction) estimated fetal weight <10th percentile, pathological Doppler ultrasound control findings

#### Placement:

port implantation can be performed only in pregnant women with anterior or anterior-lateral located placenta

#### Duration:

daily, volume of the intraumbilical infusion limited to 10% of the estimated fetoplacental blood volume per day

### AFR

#### Contraindication:

cervical insufficiency, infection, contractions

#### Purpose:

prolongation of pregnancy, reduction of perinatal morbidity and mortality

#### Usage:

long-term amnioinfusion through a subcutaneously implanted portsystem

#### Period:



between 22-30th week of gestation with PPROM until delivery to replace the loss of fluids

#### Placement:

ultrasound guided placement of catheter in the amniotic cavity

#### Duration:

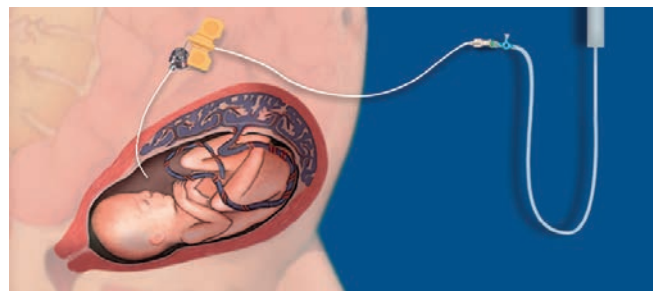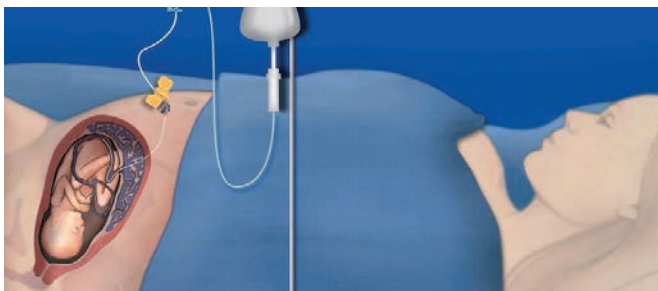continuous infusion of 50-100ml/h saline solution via portsystem

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
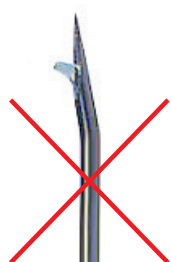
Huberkanüle

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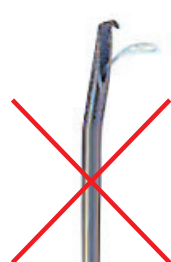# SFN®-PORT NEEDLES

non coring port needles  
with safety system

**Special needles with unique bevel for use in implantable ports and pharmaceutical pump-systems**

### Optimizes safety for the user and patient

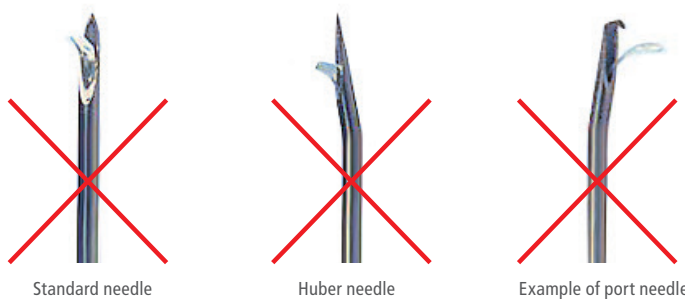
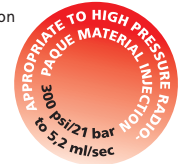
- guaranteed no punch defects when inserted into silicone membrane
- thus preventing leakage and distribution of skin and silicone particles into the bloodstream which may cause complications
- reduced infection rate
- enhances function and durability of these systems
- facilitates handling with the aid of fixation wings
- connective tubing of polyurethane equipped with Luer-lock (taxolcompatible, Latex- and PVC-free)
- stable needle tip
- protection of needle stick injuries meets requirements of TRBA 250
- easy turning down the safety clip up to the click

### Curved port cannulas

Art. No.	Canulae			PZN
	Ø mm	Length	Gauge	
SFN 0720 S	0,7	20 mm	22	07781615
SFN 0916 S	0,9	16 mm	20	07781590
SFN 0920 S	0,9	20 mm	20	06910275
SFN 0925 S	0,9	25 mm	20	07746286
SFN 0930 S	0,9	30 mm	20	07746300
SFN 1120 S	1,1	20 mm	19	07781621
SFN 1125 S	1,1	25 mm	19	07781609
SFN 1320 S*	1,3	20 mm	18	07746257
SFN 1325 S*	1,3	25 mm	18	11094033

specially designed for blood-drawing, transfusion, injection of contrast media and ascites, apheresis

\* suitable for high-pressure injection

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# SFN®-PORT NEEDLES

non-coring\_port needles straight

**Special needles with unique bevel for use in implantable ports and pharmaceutical pump-systems**

**Optimizes safety for the user and patient**

- guaranteed no punch defects when inserted into silicone membrane
- thus preventing leakage and distribution of skin and silicone particles into the bloodstream which may cause complications
- reduced infection rate
- enhances function and durability of these systems
- stable needle tip

**Straight styles for different applications**

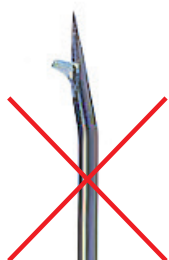
Art. No.	Canulae			PZN	
	Ø mm	Length	Gauge		
SFN 0750 G	0,7	50 mm	22	straight with Luer-Lock-Adapter without tubing	00560673
SFN 0930 G	0,9	30 mm	20	straight with Luer-Lock-Adapter without tubing	01060286
SFN 0950 G	0,9	50 mm	20	straight with Luer-Lock-Adapter without tubing	03425332
SFN 1535 G	1,5	35 mm	17	straight with Luer-Lock (optional with tubing)	10216563
SFN 1835 G	1,8	35 mm	15	straight with Luer-Lock (optional with tubing)	10838997
DPK 1835*	1,8	35 mm	15	straight with Luer-Lock (optional with tubing)	05108104
DPK 2035	2,0	35 mm	14	with mandrin + needle free adapter	02429581

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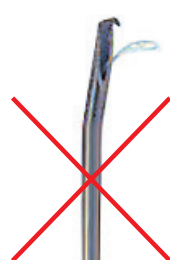
## INTRODUCER SETS PEEL-AWAY

Accessories for introducing venous catheters by Seldinger technique e.g. for implantable port catheter systems. In different sizes, depending on application and port catheter

Art. No.	article description	PZN
ITS 432006	introducer set 6 F, peel-away	02259788
ITS 432008	introducer set 8 F, peel-away	02259794
ITS 432009	introducer set 9 F, peel-away	02259848
ITS 432013	introducer set 13 F, peel-away	02260805
ITS 432015	introducer set 15 F, peel-away	08439480
ITS 432016	introducer set 16 F, peel-away	08439511
ITS 432017	introducer set 17 F, peel-away	11157785
consists of: 3 components (puncture needle, guide wire, peel-away-sheet)		

## TUNNELIZER/TROCAR

Tunnelizer/ trocars e.g. for optional subcutaneous guiding and placement of the catheter

Art. No.	Article description	Material	Length	Ø / French	PZN
T 20860	tunnelizer/ trocar	medical steel	178 mm	Ø 4 mm	06584597

# ANOKRYO®

## Basic help in treating proctologic disorders

### Indications:

- hemorrhoids I° and II°
- acute anal fissure
- perianal thrombosis

- anal discomfort
  - post-operative pain
  - itching
  - in pregnancy and lactation
  - at cryptitis, papillitis and after irradiation
  - and associated constipation

### Basic help in treating proctologic disorders

The rod reaches its lowest temperature of minus -15°C after approx. 1 – 2 hours of storage in the freezer (-15° – -20°C). The rod is then lubricated with a water soluble lubricant and is subsequently inserted into the anal canal for 5 – 10 min. After cleaning and disinfection the rod is once again stored in its case in the freezer. ANOKRYO® can be used repeatedly and contains no harmful contents, is odorless, hygienic and causes no staining.

Those treating these conditions are familiar with advantages and disadvantages of the above mentioned modes of therapy. Often a single procedure will not suffice to achieve healing. Most conditions do indeed have a tendency to heal spontaneously, this may however require a considerable amount of time, during which patients may have severe complaints.

Although the etiology of these conditions is not always clear, it appears that a permanent elevation of anal sphincter muscle tone is common to all, suggesting a contributing role in their development.

ANOKRYO® by combining low temperature with dilatation, offers an effective mode of treatment to safely and rapidly alleviate the patient's symptoms and accelerate healing.

This is achieved by cold application at temperatures of minus 15°C and dilatation to 15 mm with the ANOKRYO® rod. The duration of a single treatment is 5–10 minutes and should commence 2–3 times daily. Venous return is promoted and arterial inflow is reduced thus reducing sphincter muscle tone.

This is to relieve pain and inhibit inflammation. The use of low temperature application and dilatation in treating anal conditions is gaining popularity because it is effective, there are no untoward effects and is most suitable for home use.

Art. No.	Description	Unit	PZN
PZN 03941536	ANOKRYO low temperature rod and dilatator	1 rod + lubricant gel 30g	03941536
PZN 07470157	ANOKRYO lubricant gel 60g	1	07470157

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